

Fresenius Medical Care North America Service Bulletin

Equipment: 2008K Hemodialysis Machines

Bulletin: 10-FRK-001 Rev B

Subject: Blood Pump Serial Cable vs. Arterial Pressure Calibration (Update)

Originated By (Dept.):	Reviewed By	Approved By	Effective Date
J. Fox (TSD)	Kenneth Cogswell	CN044586	12/14/2012

Distribution: Dir TS (B. La Bier), TS Mgr (K. Cogswell), TS Trng Mgr (E. Cordova), Dir QS (M. Owen), QE (M. Mohr), Eng (C. Woods), QC Mgr (D. McKenzie) Prod Mgr (M. Singer, G. Reny), FMS TS Mgr (J. Barr), Canada (T. Greene), Mexico (J. Naranjo)

1.0 PURPOSE

To inform all Field Service and Technical Support personnel of a change in the Arterial Pressure calibration method if the Blood Pump Serial Communication cable (P/N 670658-Q) is installed between the Arterial Blood Pump and the Functional board. This bulletin affects all Functional software versions up to and including version 3.37.

2.0 PROCEDURE

With the machine turned off, disconnect the Blood Pump Serial Communication cable from the Serial Interface connector on the Arterial Blood pumps main board (LP955).

Perform the Arterial Pressure calibration as outlined in the 2008K Calibration Procedures section 2.3.1 (P/N 507296).

Note: The title for the Arterial Pressure calibration screen must say "Calibrate by Transducer". If not, check to ensure the correct cable is disconnected.

After completion of the calibration and with the machine turned off, reconnect the Blood Pump Serial Communication cable to the Serial Interface connector on the LP955 board.

If there are any questions regarding this bulletin, contact Fresenius Technical Support at 800-227-2572

© 2010 - 2012, Fresenius Medical Care North America, All rights reserved.