


2008 Series Hemodialysis Machine Operator's Manuals

Addendum for Rinsing Wands

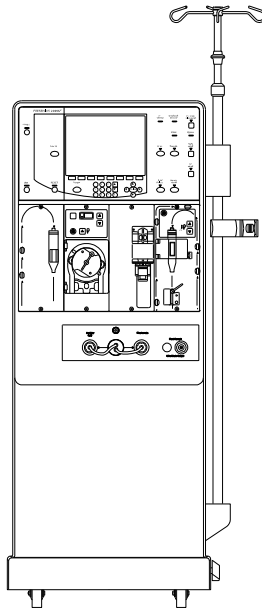

Caution: Federal (US) law restricts this device to sale only by or on the order of a physician

This is an addendum to the 2008T, 2008K, 2008K² and 2008K@home Hemodialysis Machine Operator's Manual, P/N 490122, P/N 490042, P/N 490136 and P/N 490180.

This addendum relates to the following additions:

Rinsing procedures for the Acid Concentrate (Red), Bicarbonate Concentrate (Blue), Acetic Acid (White), and Bleach (Yellow) Wands.



INSTRUCTIONS

- I. Rinse all wands following the procedure below after each use.
- II. Rinse the top of the wand assembly using RO Water as shown in Figures 1 thru 2A. Rinse the top, bottom plus the intake tube thoroughly for 30 seconds as shown.

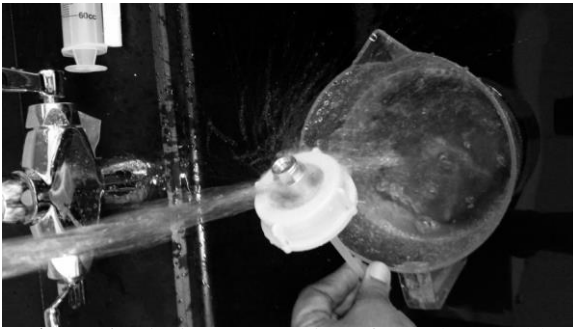


Figure 1. Rinsing Top portion of Wand Assembly for the Bleach Wand.



Figure 1A. Rinsing Top portion of Wand Assembly for the Acid Concentrate, Bicarbonate Concentrate, and Acetic Acid Wands.

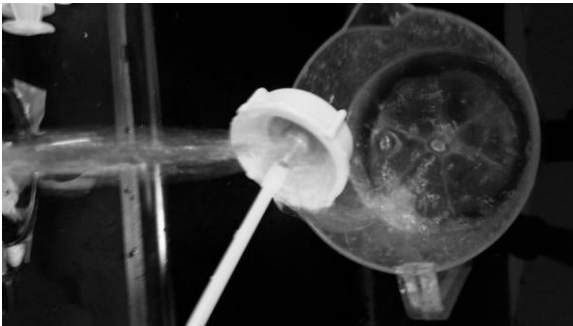


Figure 2. Rinsing Bottom portion of the Wand Assembly for the Bleach Wand.

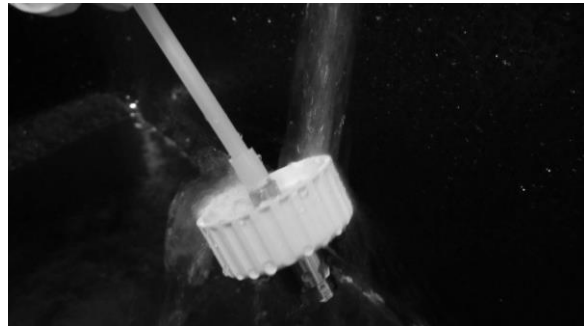


Figure 2A. Rinsing bottom portion of the Wand Assembly for the Acid Concentrate, Bicarbonate Concentrate, and Acetic Acid Wands.

- III. Place the rinsed wand assemblies in a clean area.
- IV. Visually inspect assembly for corrosion and rust ensuring that there is none inside or out.
- V. If corrosion or rust is present, repeat the rinsing procedure as shown. If the corrosion or rust is still present discard the wand assembly and replace with new one.

Please call Fresenius Medical Care Technical Support with any questions at (800) 227-2572

Distributed by:
Fresenius USA, Inc.
4040 Nelson Ave, Concord, CA 94520