

# PREVENTIVE MAINTENANCE CHECKLIST QUARTERLY/1000 HOUR

SERIAL NO. \_\_\_\_\_ DATE \_\_\_\_\_

MACHINE I.D. NO. \_\_\_\_\_ MACHINE HOURS \_\_\_\_\_

TECHNICIAN(S) \_\_\_\_\_ LABOR HOURS \_\_\_\_\_

See the Referenced Section of the 2008K Preventive Maintenance Procedures (part number 507297 Rev. L) for detailed instructions to perform the following:

Reference Section	Procedure	Value	Completed By:
2.1	Clean Filters (replace if necessary) and replace O-Rings: .....		_____
	Water Inlet.....		_____
	Acid and Bicarbonate Connectors.....		_____
	Dialysate Inline Filter.....		_____
	Dialysate Line Connectors.....		_____
2.2	Inspect Pre-UF Pump Filter.....(Replace If Necessary)		_____
2.3	Replace DIASAFE® Filter.....(If Applicable)		_____
2.4	Inspect UF Pump Check Valves.....(Replace If Necessary)		_____
2.5	Check High Voltage AC Connections.....(<0.2 ohms)	_____	_____
2.6	Calibrate UF Pump.....(Range: 24 ± 0.1cc at 24 UF Pump strokes)	_____	_____
2.7	Check Conductivity Calibration.....(Range: Δ ± 0.1mS/cm)	_____	_____
		Machine Display	External Meter
2.8	Check Temperature Calibration.....(Range: 37 ± 0.3°C)	_____	_____
		(Range: 39 ± 0.3°C)	_____
2.9	Check and Calibrate Level Detector.....		_____
2.10	Check Alarm Operation and Pressure Holding Test.....		_____
2.11	Verify pH.....(Range 6.9 to 7.6)	_____	_____
2.12	Check Power Failure Alarm.....		_____
2.13	Inspect Blood Pump Rotor.....		_____
2.14	Final Checks.....		_____

**NOTES:**

---



---



---



---